**SOFT (HYDROPHILIC) CONTACT LENSES**  
**Directions for Use**

**LENTILLES CORNÉENNES SOUPLES (HYDROPHILES)**  
**Directives**

EN: .....2  
FR: .....7

EN:

## SOFT (HYDROPHILIC) CONTACT LENSES Directions for Use

**Important:** Please read carefully and keep this information for future use. This package insert is intended for the eye care practitioner but should be made available to patients upon request. The eye care practitioner should provide the patient with the patient instructions that pertain to the patient's prescribed lenses.

Please refer to the carton for: device name, contents, identifier, lot number, and expiration date.

### SYMBOLS KEY:

The following symbols may appear on the label or carton.

SYMBOL	DEFINITION
	Caution: Federal (USA) law restricts this device to sale by or on the order of a licensed practitioner
	Caution
	Use by Date (expiration date)
	Batch Code
	Sterile using Steam Heat
	Single sterile barrier system
	Original Manufacturer
	Authorized representative in the European Community
	Do not use if package is damaged
	Consult instructions for use / consult electronic instructions for use
	Do not re-use
	Date of manufacture
	Signifies European conformity (CE) mark
	Indication of a Medical Device
	Denotes UV Blocking

The Private Label Licence Holder is Specsavers Canada Inc (Legal Manufacturer) and the Original Manufacturer of the Medical Device is CooperVision.

### INDICATIONS FOR USE:

#### **Multifocal (stenfilcon A)**

- The **MULTIFOCAL SOFT CONTACT LENSES** are indicated for the correction of refractive ametropia (myopia and hyperopia) and emmetropia with presbyopia in aphakic and non-aphakic persons with non-diseased eyes. The lenses may be worn by persons who exhibit astigmatism of -2.00 diopters or less that does not interfere with visual acuity.

#### **DAILY DISPOSABLE WEAR (stenfilcon A)**

- The **SOFT (HYDROPHILIC) CONTACT LENSES** are daily wear single use contact lenses.

## CONTRAINDICATIONS (REASONS NOT TO USE)

**Do not use the lens when any of the following conditions exist:**

- Acute and subacute inflammation or infection of the anterior chamber of the eye.
- Any eye disease, injury, or abnormality that affects the cornea, conjunctiva, or eyelids.
- Severe insufficiency of lacrimal secretion (dry eyes).
- Corneal hypoesthesia (reduced corneal sensitivity), if not aphakic.
- Any systemic disease that may affect the eye or be exaggerated by wearing contact lenses.
- Allergic reactions of ocular surfaces or adnexa that may be induced or exaggerated by wearing contact lenses or use of contact lens solutions.
- Allergy to any ingredient, such as mercury or thimerosal, in a solution, which is to be used to care for any lens.
- Any active corneal infection (bacterial, fungal, or viral).
- If eyes become red or irritated.
- The patient is unable to follow lens care regimen or unable to obtain assistance to do so.

## WARNINGS

**Patients should be advised of the following warnings pertaining to contact lens wear:**

- **PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN SERIOUS INJURY TO THE EYE.** It is essential that patients follow their eye care practitioner's directions and all labeling instructions for proper use of lenses. Eye problems, including corneal ulcers, can develop rapidly and lead to **loss of vision**. Daily wear lenses are not indicated for overnight wear, and patients should be instructed not to wear lenses while sleeping. Clinical studies have shown that the risk of serious adverse reactions is increased when these lenses are worn overnight. Studies have shown that contact lens wearers who are smokers have a higher incidence of adverse reactions than nonsmokers. If a patient experiences eye discomfort, excessive tearing, vision changes, or redness of the eye, the patient should be instructed to immediately remove lenses and promptly contact his or her eye care practitioner.

## PRECAUTIONS

### Special Precautions for Eye Care Practitioners

- Due to the small numbers of patients enrolled in clinical investigation of lenses, all refractive powers, design configurations, or lens parameters available in the lens material are not evaluated in significant numbers. Consequently, when selecting an appropriate lens design and parameters, the eye care practitioner should consider all characteristics of the lens that can affect lens performance and ocular health, including oxygen permeability, wettability, central and peripheral thickness, and optic zone diameter.
- The potential impact of these factors on the patient's ocular health should be carefully weighed against the patient's need for refractive correction; therefore, the continuing ocular health of the patient and lens performance on the eye should be carefully monitored by the prescribing eye care practitioner.
- Patients who wear contact lenses to correct presbyopia may not achieve the best corrected visual acuity for either far or near vision. Visual requirements vary with the individual and should be considered when selecting the most appropriate type of lens for each patient.
- Aphakic patients should not be fitted with any contact lenses until the determination is made that the eye has healed completely.

- Fluorescein, a yellow dye, should not be used while the lenses are on the eyes. The lenses absorb the dye and become discolored. Whenever fluorescein is used in the eyes, the eyes should be flushed with a sterile saline solution that is recommended for in-eye use.
- Before leaving the eye care practitioner's office, the patient should be able to promptly remove the lenses or should have someone else available who can remove the lenses for him or her. Eye care practitioners should instruct the patient to remove the lenses immediately if the eye becomes red or irritated.

**Eye care practitioners should carefully instruct patients about the following safety precautions:**

- Do not use saliva or anything other than the recommended solutions for lubricating or wetting lenses.
- If the lens sticks (stops moving) on the eye, follow the recommended directions on Care for a Sticking Lens. The lens should move freely on the eye for the continued health of the eye. If non-movement of the lens continues, the patient should be instructed to **immediately** consult his or her eye care practitioner.
- Always wash and rinse hands before handling lenses. Do not get cosmetics, lotions, soaps, creams, deodorant, or sprays in the eyes or on the lenses. It is best to put on lenses before putting on makeup. Water-based cosmetics are less likely to damage lenses than oil-based products.
- Do not touch the contact lenses with the finger or hands if the hands are not free of foreign materials, as lens damage may occur.
- Always handle lenses gently and avoid dropping them.
- Never use tweezers or other tools to remove lenses from the lens container unless specifically indicated for that use. Pour lens into your hand.
- Do not touch the lens with fingernails.
- Never wear lenses beyond the period recommended by the eye care practitioner.
- Always discard disposable lenses after the recommended wearing schedule prescribed by the eye practitioner.
- Avoid all harmful or irritating vapors and fumes while wearing lenses.
- If aerosol products such as hairspray are used while wearing lenses, exercise caution and keep eyes closed until the spray has settled.
- Ask the eye care practitioner about wearing the lenses during sporting activities.
- Inform the doctor (health care practitioner) about being a contact lens wearer.
- Always contact the eye care practitioner before using any medicine in the eyes.
- Always inform the employers of being a contact lens wearer. Some jobs may require use of eye protection equipment or may require that the patient not wear contact lenses.
- As with any contact lens, follow-up visits are necessary to assure the continuing health of the patient's eyes. The patient should be instructed as to a recommended follow-up schedule.

**ADVERSE REACTIONS**

The patient should be informed that the following problems may occur:

- Eyes stinging, burning, or itching (irritation), or other eye pain.
- Comfort is less than when the lens was first placed on the eye.
- Feeling that something is in the eye such as a foreign body or a scratched area.
- Excessive watering (tearing) of the eyes.
- Unusual eye secretions.
- Redness of the eyes.
- Reduced sharpness of vision (poor visual acuity).

- Blurred vision, rainbows, or halos around objects.
- Sensitivity to light (photophobia).
- Dry Eyes

If the patient noticed any of the above, he or she should be instructed to:

- **Immediately remove the lenses.**
- If the discomfort or the problem stops, then look closely at the lens. If the lens is in some way damaged, do not put the lens back on the eye. Place the lens in the storage case and contact the eye care practitioner. If the lens has dirt, an eyelash, or other foreign body on it, or the problem stops and the lens appears undamaged, the patient should thoroughly clean, rinse, and disinfect both lenses; then reinsert them. After reinsertion, if the problem continues, the patient should **immediately remove the lenses and consult the eye care practitioner.**

When any of the above problems occur, a serious condition such as infection, corneal ulcer, neovascularization, or iritis may be present. The patient should be instructed **to keep the lens off the eye and seek immediate** professional identification of the problem and prompt treatment to avoid serious eye damage.

### **FITTING**

Conventional methods of fitting contact lenses apply to all **SOFT CONTACT LENSES**. For a detailed description of the fitting techniques, refer to the **SOFT CONTACT LENSES** Professional Fitting and Information Guide, copies of which are available from:

Specsavers Canada Inc  
Suite 101-3001 Wayburne Drive  
Burnaby, BC, CA, V5G 4W3  
Toll Free 1-800-705-9815  
[www.specsavers.ca](http://www.specsavers.ca)

### **WEARING SCHEDULE**

**The wearing schedule should be determined by the eye care practitioner.** Patients tend to over-wear the lenses initially. The eye care practitioner should emphasize the importance of adhering to the initial maximum wearing schedule. Regular check-ups, as determined by the eye care practitioner are also extremely important.

#### **Daily Disposable Wearing Schedule:**

It is recommended that all lenses be discarded and replaced with a new lens on a daily basis. Do not wear lenses during sleeping hours.

**DAILY DISPOSABLE WEAR:** (less than 24 hours, while awake)

The maximum suggested wearing time is 12 hours.

### **LENS CARE DIRECTIONS**

Eye care practitioners should review with the patient lens care directions, including basic lens care information.

- Always wash, rinse, and dry hands before handling contact lenses.
- Do not use saliva or anything other than the recommended solutions for lubricating or rewetting. Do not put lenses in the mouth.
- The patient should always have a spare pair of lenses at all times.

### **General Lens Care (For Daily Disposable / Daily Wear Single Use Lenses)**

- The eye care practitioner should review with the patient that no cleaning or disinfecting is needed with disposable lenses.
- The patient should always have a spare pair of lenses at all times.

### **CARE FOR A DRIED OUT (DEHYDRATED) LENS**

If any SOFT CONTACT LENS is exposed to air while off the eye, it may become dry and brittle. In this event, simply dispose of the lens and replace with a fresh one.

### **CARE FOR A STICKING (NONMOVING) LENS**

If the lens sticks (stops moving or cannot be removed), the patient should be instructed to apply 2 to 3 drops of the recommended lubricating or rewetting solution directly to the eye and wait until the lens begins to move freely on the eye before removing it. If non-movement of the lens continues more than 5 minutes, the patient should immediately consult the eye care practitioner.

### **EMERGENCIES**

The patient should be informed that if chemicals of any kind (household products, gardening solutions, laboratory chemicals, etc.) are splashed into the eyes, the patient should: **FLUSH THE EYES IMMEDIATELY WITH TAP WATER AND IMMEDIATELY CONTACT THE EYE CARE PRACTITIONER OR VISIT A HOSPITAL EMERGENCY ROOM WITHOUT DELAY.**

### **HOW SUPPLIED**

Each lens is supplied sterile in a blister or vial containing buffered saline solution. The blister or vial is labeled with the base curve, diameter, dioptric power, manufacturing lot number, and expiration date of the lens.

### ***DO NOT USE IF THE BLISTER PACK IS BROKEN OR THE SEAL HAS BEEN DAMAGED***

### **REPORTING OF ADVERSE REACTIONS**

All serious adverse experiences and adverse reactions observed in patients wearing any SOFT CONTACT LENS or experienced with the lenses should be reported to:

Specsavers Canada Inc  
Suite 101-3001 Wayburne Drive  
Burnaby, BC, CA, V5G 4W3  
Toll Free 1-800-705-9815  
[www.specsavers.ca](http://www.specsavers.ca)

### **PRODUCT MANUFACTURING LOCATIONS**

#### **PRODUCTS MADE IN UK**

***Manufacturing location: Hamble, SO31 4RF, UK***

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